



RE-USE OF THE DEVICE

The PROGUARD™ gloves can be resterilized 4 times by Ethylene Oxide.

BEFORE ITS RE-USE, THE USED GLOVE MUST UNDERGO THE FOLLOWING PROCEDURE:

1. Cleaning and disinfection:

- Accurately wash the gloves under running water.
- If necessary, use a suitable detergent / disinfectant solution, such as HSEMOSOL or PROTEZIM to remove any blood or organic residues.
- Rinse under running water and eliminate all traces of the solution used.
- If possible, hang the gloves and let them drip dry at room temperature in a clean place, away from light and heat sources.

2. Re-packaging:

- Carry out in a clean and controlled area, with low particle and microbial contamination.
- Re-package the gloves introducing them, in pairs, in envelopes suitable for Ethylene Oxide.
- Sterilization. Seal each package in the most suitable way.

3. Re-sterilization:

- Ethylene Oxide re-sterilization must be carried out by authorized companies after their system has been validated, in order to guarantee the method's reliability and, therefore, the sterility of the device after each procedure.