



## USE & CARE OF BAR-RAY PROTECTIVE APPAREL

*Thank you for choosing Bar-Ray, the global leader in radiation protection.*

*To ensure the longevity of your new garment, please follow these simple guidelines:*

- *Radiograph the garment upon receipt and then regularly afterward to insure that no damage has occurred to the protective core.*
- *Clean your garment in accordance with your facilities infection control protocols.*
- *Spot cleaning can be done with mild soap and a damp cloth or NON-BLEACH containing wipes.*
- *Hang your garment properly by the shoulders or hanging loops.*
- *Do not fold or crease the garment.*
- *Hook and loop fasteners should be matched together when hanging so they do not snag.*
- *Do not expose the garment to extreme temperatures, submerge in liquids or place in a dryer.*
- *Do not lean or rub against sharp corners/edges or store sharp objects in the pockets of your garment.*

### WARRANTY

*Bar-Ray guarantees its products against defects in materials or workmanship for 2 years from date of factory shipment.*

### QUESTIONS

*Please call us or email us at [info@bar-ray.com](mailto:info@bar-ray.com)*